



# ABSORBENT PADS

## Instruction for use

The pad for additional bed protection is designed to be reused after washing. It is suitable for home and institutional use.

### Why use the pad?

The pad forms a waterproof protection for the bed and is gentle on the user. The top layer of soft plush provides a soft and pleasant touch to the skin. The unique construction of the fabric ensures that unwanted liquid is quickly drained into the middle layers of the pad, which retain it and prevent it from seeping back into the top layer that is in contact with the body. The fluid is removed from the pad during the next wash.

The rounded corners of the pad reduce the risk of wear and tear during machine washing and tumble drying, thus extending its life. The product variant with tabs allows the pad to be firmly fixed on the bed. The product variant with handles allows easier handling when positioning on the bed.

**The pad is not intended and must not be used for carrying or lifting persons! It is intended for positioning on the bed only!**

### 4L

Four-layer incontinence pad - 100-200 ml/4 hours.

### 5L

Five-layer incontinence pad - 200-300 ml or more/4 hours.

### Composition:

The pad is made of multi-layer laminated fabric bonded by flatstitching and lamination. Five-layer fabric with a weight of 512 g/m<sup>2</sup>, four-layer fabric with a weight of 440 g/m<sup>2</sup>.

### Top layer:

Warp plush knitted fabric made of 100% polyester fibres micro

### Middle/ absorbent layer:

fleece/felt made of 100% polyester fibres (1x/four-layer, 2x/five-layer)

### Bottom layer:

warp knitted fabric made of 100% polyester fibres, with a laminated polyurethane membrane, forming a waterproof barrier.

### How to proceed with use?

Place the pad on the user's bed, in the exposed area, with the plush layer on top (towards the body) and the waterproof layer on the bottom (towards the bed mattress).

For the tabbed version of the product, slide the tabs under the mattress (this fixes the pad firmly in place). Wash the mattress pad after each use (saturation) of the middle layers with liquid.

### Laundry procedure:

At a temperature up to 95 °C, with normal detergents (pre-wash, intermediate spin and spin programme). The lifetime of the pad is approx. 300 washing and drying cycles.

**Please note: It is not recommended to use detergents with bleaching effect, bleach the pad, dry clean, use fabric softener or iron (it will damage the functional properties of the product).**



# ABSORBENT PADS

## Contraindications:

No side effects such as allergies, increased soreness through the formation of bedsores, or any other contraindications have been described when using the absorbent pad. Mild skin irritation (itching) in hypersensitive persons is not a problem.

## Ecological disposal of the product:

At the end of the product lifetime, wash the product and dispose of it in separated waste. Used (unwashed product) must be treated as microbiologically contaminated material!

## Storing:



Store in a dry place at a temperature of 5 -25 °C, do not expose to direct sunlight.




CLINITEX s.r.o.  
Vratimovská 672/42,  
Ostrava Kunčičky 718 00  
www.clinitex.cz

Absorption pad 5L



003 ABSO5L  
100% POLYESTER  
MEMBRANE-POLYURETHAE  
WEUGGT 515 gr/qm  
300x PRANÍ A SUŠENÍ

!! IS NOT INTENDED FOR AND  
MUST NOT BE USED ON  
CARRYING AND LIFTING PERSONS!  
IS INTENDED FOR POSITIONING ON  
THE BED ONLY! DO NOT USE FABRIC  
SOFTENER!

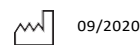

LOT 20140906

## The meaning of the symbols on the label:

	Washable at temperatures up to 95 °C		Manufacturer
	No bleaching		Date of manufacturing
	No ironing		Read the instructions
	Prohibition of dry-cleaning		Catalogue number
	Tumble drying, lower drying temperature		Store in dry conditions
5 Layers			Protect against the sun
4 Layers			Temperature limits
Do not use fabric softener			Packaging symbols, disposal use



## The Manufacturer (and distributor):

CLINITEX S.R.O., VRATIMOVSKÁ 672/42  
718 00 OSTRAVA – KUNČIČKY, ČESKÁ REPUBLIKA